Disinfection Wipes

13,5x18,5 CM

- (GB)** Disinfection wipes for surfaces of non-invasive medical devices, including surfaces sensitive to alcohol. Specially formulated for use in the medical and dental environment. Contains detergent. Using a new wipe for each application wipe objects to be disinfected so that they are completely moistened. Let the disinfectant take action over a period of 30 seconds (to achieve full efficacy leave for 5 minutes). Allow surface to dry or wipe with clean tissue. For professional use only.
- Effective spectrum:** Bactericidal (30 secs). MRSA, VRE (60 secs). Legionella pneumophila (5 min). Yeasticide (30 secs). Fungicidal (30 secs). Clostridium difficile (5 min). Mycobacterium avium (2 min). Virucidal activity against enveloped viruses (2 min): Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus (HDV), HIV, Influenza Virus, Corona Virus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies. Measles. Tested to: EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.
 - Composition:** Aqua, Benzyl-C12-18-alkyldimethyl Chlorides, C12-14-Alkyl [(Ethylphenyl) Methyl] Dimethyl Chloride, Didecylmethylammonium Chloride, 2-Propanol (not an active, <1%), Surfactant, Dipropylene glycol monomethyl ether, Polyhexamethylene biguanide hydrochloride, Cocamidopropyl Betaine, Tetrasodium EDTA, Antifoam, Preservative, Aroma.
 - Warning:** P101: If medical advice is needed, have product container or label at hand. P102: Keep out of reach of children. P301+P312: If SWALLOWED: Call a POISON CENTER or doctor/pharmacist if you feel unwell. P305+P351+P338: If IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. The wearing of protective gloves is recommended. **Storage:** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications:** Incompatible materials: Aluminium, copper. **Disposal:** Do not discharge liquid into drains or rivers. Do not flush. Not maceratable. Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

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Topdental (Products) Ltd
DIRECTA AB